

Verd Orthopaedic Seating



Operating Instructions:

1. Seat height adjustment: Lift lever (1) to raise and lower seat height.
2. Seat angle adjustment: Lift lever (2) to alter seat positions, push lever down to lock in desired position. Leaving lever in “up” position will allow “free float” option, this operates in conjunction with or independently of Back Angle Adjustment.
3. Back Angle Adjustment: Lift lever (3) to adjust back angle, release lever to lock.
4. Body Weight Tension Adjustment: When seated wind knob (4) clockwise to loosen and anti-clockwise to tighten.
5. Seat Slide Adjustment (If fitted): Lift lever (5) and slide either backwards or forwards to increase or decrease seat depth. Lever is located on left front under seat, when seated.
6. Back Height Adjustment (6): Raise the back manually to desired height. Ratchet mechanism will lock in several height positions. When ratchet reaches the top it will release to the lowest setting.
7. Inflatable Lumbar/Sacrum/Thoracic Support (if Fitted). Pump bulb to required level. To deflate press valve button on neck of bulb.
8. Headrest (if fitted) pull headrest gently from housing to increase height. Adjust tilt and angle to suite
9. Arms (If Fitted). Depress Button on outside of arms to raise or lower. On ADJ08 (shown) Arm pads slide forward and rearwards.

Returns will only be accepted if chair is re-boxed.

Product Help-line: 01388 748 999